

# SIEMENS MAGNETOM TrioTim syngo MR B17

\\USER\Siemens\_3T\_TrioTim\_VB17\_HR\ADNI2\_HR\HUMAN\MPRAGE

TA: 9:14    PAT: Off    Voxel size: 1.0x1.0x1.2 mm    Rel. SNR: 1.00    SIEMENS: tfl

### Properties

Prio Recon	Off
Before measurement	
After measurement	
Load to viewer	On
Inline movie	Off
Auto store images	On
Load to stamp segments	Off
Load images to graphic segments	Off
Auto open inline display	Off
Start measurement without further preparation	On
Wait for user to start	Off
Start measurements	single

### Routine

Slab group 1	
Slabs	1
Dist. factor	50 %
Position	Isocenter
Orientation	Sagittal
Phase enc. dir.	A >> P
Rotation	0.00 deg
Phase oversampling	0 %
Slice oversampling	0.0 %
Slices per slab	176
FoV read	256 mm
FoV phase	93.8 %
Slice thickness	1.20 mm
TR	2300 ms
TE	2.98 ms
Averages	1
Concatenations	1
Filter	Prescan Normalize
Coil elements	HEA;HEP

### Contrast

Magn. preparation	Non-sel. IR
TI	900 ms
Flip angle	9 deg
Fat suppr.	None
Water suppr.	None
Averaging mode	Long term
Reconstruction	Magnitude
Measurements	1
Multiple series	Off

### Resolution

Base resolution	256
Phase resolution	100 %
Slice resolution	100 %
Phase partial Fourier	Off
Slice partial Fourier	Off
Interpolation	Off
PAT mode	None
Matrix Coil Mode	Auto (CP)
Image Filter	Off
Distortion Corr.	Off
Unfiltered images	Off
Prescan Normalize	On
Normalize	Off
B1 filter	Off

Raw filter	Off
Elliptical filter	Off

### Geometry

Multi-slice mode	Single shot
Series	Interleaved
Set-n-Go Protocol	Off
Table position	H
Table position	0 mm
Inline Composing	Off

### System

Body	Off
HEP	On
HEA	On
Positioning mode	REF
MSMA	S - C - T
Sagittal	R >> L
Coronal	A >> P
Transversal	F >> H
Save uncombined	Off
Coil Combine Mode	Adaptive Combine
AutoAlign	---
Auto Coil Select	Default
Shim mode	Standard
Adjust with body coil	Off
Confirm freq. adjustment	Off
Assume Silicone	Off
? Ref. amplitude 1H	0.000 V
Adjustment Tolerance	Auto
Adjust volume	
Position	Isocenter
Orientation	Sagittal
Rotation	0.00 deg
F >> H	256 mm
A >> P	240 mm
R >> L	212 mm

### Physio

1st Signal/Mode	None
Dark blood	Off
Resp. control	Off

### Inline

Subtract	Off
Std-Dev-Sag	Off
Std-Dev-Cor	Off
Std-Dev-Tra	Off
Std-Dev-Time	Off
MIP-Sag	Off
MIP-Cor	Off
MIP-Tra	Off
MIP-Time	Off
Save original images	On

### Sequence

Introduction	On
Dimension	3D
Elliptical scanning	Off
Asymmetric echo	Off
Bandwidth	240 Hz/Px
Flow comp.	No

# SIEMENS MAGNETOM TrioTim syngo MR B17

Echo spacing	7.1 ms
--------------	--------

---

RF pulse type	Fast
---------------	------

Gradient mode	Normal
---------------	--------

Excitation	Non-sel.
------------	----------

RF spoiling	On
-------------	----